

# Avogel<sup>®</sup>

HYDROGEL SHEETING FOR SCARS

## Effective treatment for hypertrophic and keloid scars.

- Clinically proven to improve scar appearance.
- Decreases pain and itch.
- Hypoallergenic; contains no silicone.
- Cost effective.

Avogel Hydrogel Sheeting is an adhesive sheet that stays in place and can be cut to size. It works by providing a barrier to prevent heat and water loss through the skin.



Polymer Technologies, Inc.

©2009 Avocet and Avogel are registered trademarks. Made in the USA Ver. PP6-06

©2009 Avocet Polymer Technologies, Inc. 12/09

# Avosil<sup>™</sup>

SCAR CARE OINTMENT

## Reduces formation of scar tissue.

- Improves the appearance of scars resulting from surgery, burns, and injury.
- Soothes itchy and tender scars.
- Moisturizes and softens scars.

Avosil Scar Care Ointment is a topical formulation for scars, containing sodium salicylate. It can be applied 2-3 times a day. Avosil reduces inflammation, which controls pain, tenderness and itching. The safety of Avosil's components is well established.

**FOR MORE INFORMATION**  
 Visit [www.avocetcorp.com](http://www.avocetcorp.com)  
 or call toll free:  
**866-352-SCAR (7227)**

### INSURANCE COVERAGE

Avogel is covered by most insurance plans, if prescribed by a physician. Please call 866-352-SCAR (7227) for details.

R <sub>x</sub>	PATIENT ALLERGIES		PATIENT AGE	DATE
	PATIENT NAME			
	NO. & STREET			
	CITY		STATE	

DRUG NAME/DESCRIPTIONS	SIZE	QUANTITY	REFILLS
<b>Avosil<sup>™</sup></b>	<input type="checkbox"/> 2 oz.		<input type="checkbox"/> PRN <input type="checkbox"/> NONE
	<input type="checkbox"/> 4 oz.		<input type="checkbox"/> TIMES
	<input type="checkbox"/> 12 oz.		___ WITHIN ___ MONTHS

DRUG NAME/DESCRIPTIONS	SIZE	QUANTITY	REFILLS		
<b>Avogel<sup>®</sup></b>	<input type="checkbox"/> 4 x 4		<input type="checkbox"/> PRN <input type="checkbox"/> NONE		
	<input type="checkbox"/> 8 x 8			<input type="checkbox"/> TIMES	
	<input type="checkbox"/> 6 x 48				___ WITHIN ___ MONTHS
	<input type="checkbox"/> 4 x 4 TRIPLE				
<input type="checkbox"/> 8 x 8 TRIPLE					


	FB	CB
M.D.	DISPENSE DATE	

# DIRECTIONS: Avogel and Avosil



1.

1. When directed by your physician, spread a 1/16" thick layer of Avosil to your scar and gently rub in.

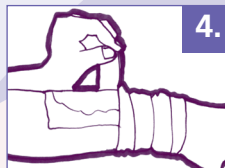

4.

4. Secure the Avogel in place with netting, gauze dressing, elastic bandage material or a pressure garment.

If using Avogel also (steps 2-6)




2.

2. Trim the Avogel sheet so that it extends at least 1" around your scar.




5.

5. If you are reusing, wash the Avogel after each use with mild soap (not dish soap), rinse and dry.

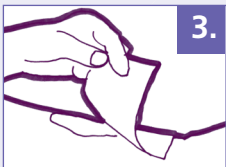

3.

3. Make sure Avogel is absorbed, wait 1-2 hours. Gently press the adhesive side of the Avogel over your scar. Wear the sheeting for 12-18 hours.

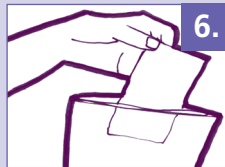

6.

6. Store Avogel in a clean, re-sealable bag at room temperature.

## ADDITIONAL TIPS

1. Many patients find that the best time to use Avogel is after work, and overnight, when they are less active.
2. Wash and dry your hands and the scar before applying Avosil and Avogel.
3. After removing Avogel, keep the skin open to air for 6 – 8 hours.

3. Discontinue use and contact your physician if signs of infection occur (increased pain, swelling, redness, or drainage), if a rash or odor develops, or any other unexpected symptoms occur.
4. Do not wear Avogel while swimming or bathing.
5. Remove and discard Avogel if it becomes hard or soiled.

## PRECAUTIONS

1. Avosil is not to be used by children with flu-like symptoms.
2. If allergic to aspirin, do not use Avosil without consulting your physician.

## INSURANCE COVERAGE

Avogel is covered by most insurance plans, if prescribed by a physician. Please call 866-352-SCAR (7227) for details.

Avogel® is FDA-approved for use in the management of hypertrophic scars. Avosil™ is formulated to help the appearance of scars. Pharmacists can order Avogel and Avosil directly from Avocet Polymer Technologies by calling toll free (866) 352-SCAR (7227). Avocet's products are available through Cardinal (800) 530-2696 and McKesson (800) 482-DRUG via drop-ship.

Product NDC #	Product Description	Cardinal ID #	McKesson Econo #
<b>Avogel Hydrogel Sheeting</b>			
76170-0108-81	Avogel 8" x 8" 1 single pack	3609583	1434992
76170-0128-81	Avogel 8" x 8" 12 single packs		
76170-0108-83	Avogel 8" x 8" 1 triple pack	3764560	1442086
76170-0128-83	Avogel 8" x 8" 12 triple packs		
76170-0104-43	Avogel 4" x 4" 1 triple pack	3609591	1678937
76170-0124-43	Avogel 4" x 4" 12 triple packs		
76170-0106-48	Avogel 6" x 48" 1 single roll	3609609	1430776
76170-0164-83	Avogel 6" x 48" 3 rolls		1438472
<b>Avosil Scar Ointment</b>			
76170-0200-01	Avosil Scar Care Gel, 2 oz	3737194	1927003
76170-0200-20	Avosil Scar Care Gel, Box of 20		
76170-0400-01	Avosil Scar Care Gel, 4 oz	3609559	1445576
76170-0400-20	Avosil Scar Care Gel, Box of 20		
76170-1200-01	Avosil Scar Care Gel 12 oz	3764545	1927417
76170-1200-06	Avosil Scar Care Gel, Box of 6		



FOR MORE INFORMATION  
Visit [www.avocetcorp.com](http://www.avocetcorp.com)  
or call toll free: 866-352-SCAR (7227)